

PRIDE - Curtis/Q-Logic


TO BE USED WITH VERSIONS JANUARY 2012 TO CURRENT



Program Adjustments

SELECT

Top Right Button of Programmer

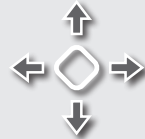

Use Directional Arrows to Navigate Menu



Arrow Advances in the Menu



Arrow Backs you out of the Menu



To change preferences, you may need to go into Idle

Main Menu



AUX Modes
WILL ONLY APPEAR IF AN ECU IS CONNECTED



Hand Control Settings



Attendant Joystick



SPECIALITY CONTROLS







After Program Adjustments is Selected





Highlighted boxes indicate recommended changes

Profile Setup	P1	P2	P3	P4	P5	CHOOSE PREFERENCE
MODE OPTIONS	Drive, Seat Program, Aux Program, Rest, Disable					
Profile Type	P1-P5, Text icon, Shape					
Text & Icon	Talk, TV, Sport, Ramp, Work, School, Seat, Fast, Slow, Outdoor, Indoor					
Shapes	Circle, Penta, Square, Triangle, Cross, Star					
DRIVE	P1	P2	P3	P4	P5	




Pride - Curtis / Q-Logic (continued)

DRIVE (CONTINUED)	P1	P2	P3	P4	P5
<div style="border: 1px dashed black; padding: 5px; width: fit-content;"> <p>SPEED SETTINGS: In Digital, Speed Turn Ratio should be set 10% less than</p> </div>	<p style="text-align: right;">CHOOSE PREFERENCE</p>  <p>FwdSp Fwd Acc Fwd Dec TrnSp Trn Acc Trn Dec Spd Trn Rate Trn Sens Rev Sp Rev Acc Rev Dec</p> <hr style="border-top: 1px dotted black;"/> <p>Latch Fwd Latch Rv Power</p>				
ACTIVE DEVICE	3-SWITCH HEAD				
 <p>3-SWITCH HEAD Folder (for ASL Head Array)</p>	 Back Switch Toggle Folder				
	m: Back Toggle				DISABLED
	m: De-Bounce Feedback				On
	m: Back Toggle De-Bounce				500 ms
	m: Back Toggle Command				500 ms
	m: Latch Toggle Auto Flip				Off
	m: Toggle While Driving				Disabled
	m: Toggle in IR				Disabled
	m: Toggle in ECU				Disabled
	 Device Options Folder				
	m: Device Double Command				DISABLED
	m: Seat Head Left / Right Swap				No
	ASL Timing				50
	m: Device Double Command Time				700 ms
	m: Device long Command Time				1000 ms

Pride - Curtis / Q-Logic (continued)

ACTIVE DEVICE	3-SWITCH HEAD (Continued)	
 <p>3-SWITCH HEAD Folder (for ASL Head Array)</p>	 Switch Options Folder	
	m: Mode Jack Switch Type = Toggle	TOGGLE
	m: Mode Key Navigation	Simple
	m: Mode Jack Supervision	No
	m: On / Off Switch Type	Disabled
	m: On / Off Jack Supervision	No
	m: Jack Double Command	1000 ms
	m: Jack Long Command	1300 ms
	m: D-Sub-9 Supervision	No
PROFILE CHANGE STOP Enabled		
START UP PROFILE		

**Main Menu
(continued)**

-  **Enhanced Display**
-  **Seat**
-  **System Settings**