


ASL 151 PG INTERFACE

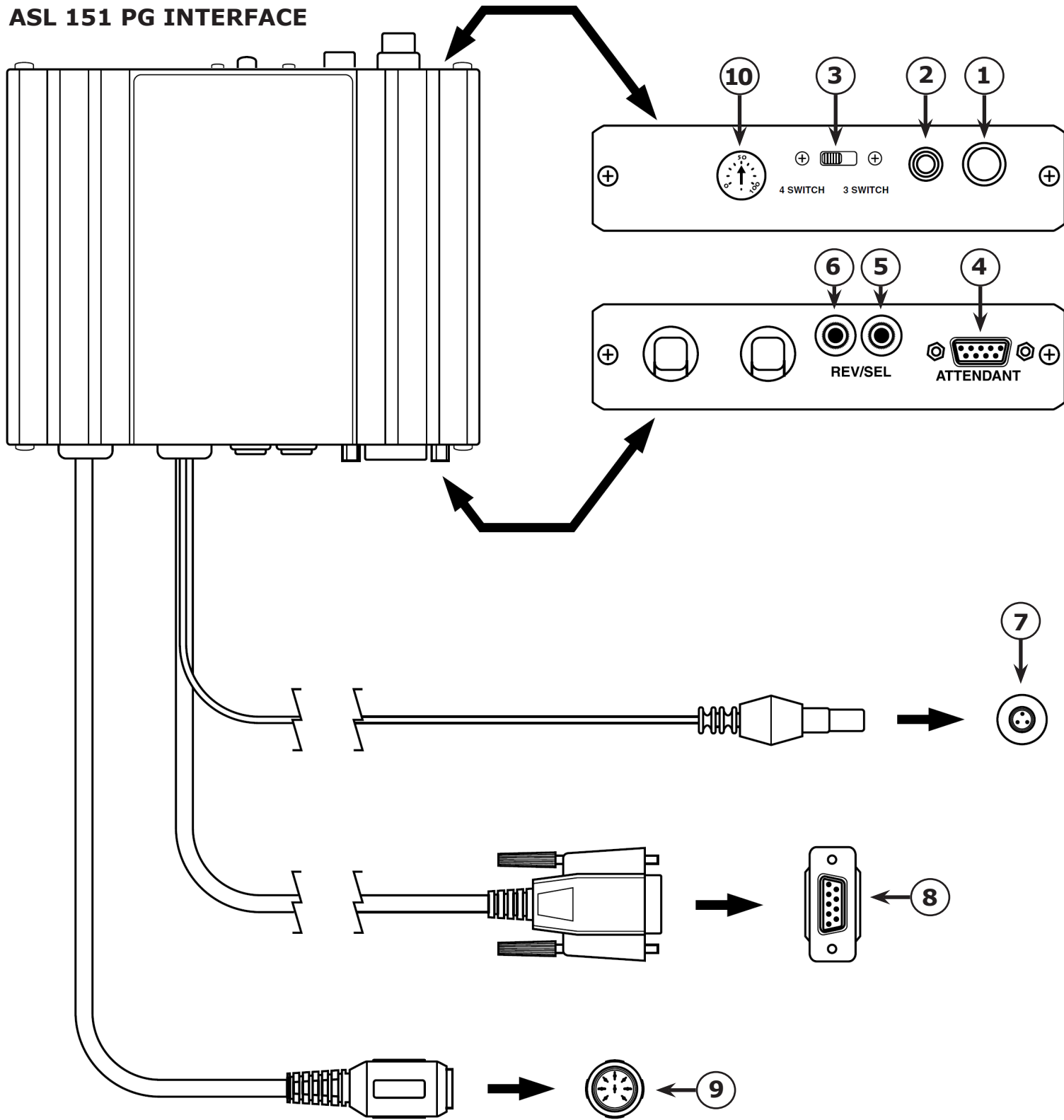

1) Power Button

2) Power Indicator Light

3) Slide Switch

3 Switch: Fourth Switch on Round 8-Pin = Reset / Mode Change
 4 Switch: Fourth Switch on Round 8-Pin = Reverse

4) Digital Attendant Port(ASL Digital Attendant Only)

5) 1/8" Mono Input Port for Mechanical Switch to Activate Reset / Mode Change

6) 1/8" Mono Input Port for Mechanical Switch to Activate Reverse (Does Not Work in 3-Switch Programming)

7) 3-Pin Input Port for Electronic Switch to Activate Reset / Mode Change

8) Power Chair Accessory Port Connector / Input to Chair

9) 8-Pin Connector to ASL Digital Driver Control

10) Potentiometer: Adjustable Timer in 3-Switch Programming that will replace the "Double Tap" with a "Press and Hold" of the Reset Switch. The Potentiometer Adjusts the Timing from 1 Second (Pot Set to 20) to 7 Seconds (Pot Set to 100).