





ATOM 558 Mouse Emulator Common Setups

TECHNICAL USER GUIDE

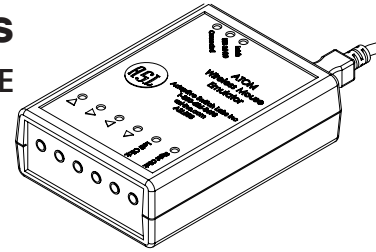


SETTING UP YOUR DEVICE

There are Four (4) Common Setups for the ASL 558 ATOM Wireless Mouse Emulator.

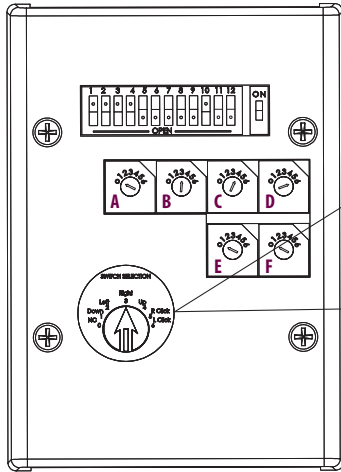


ASL 558 Wireless Switch Emulator (Bottom View)

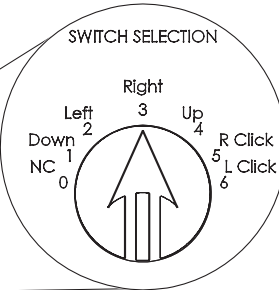


MOUSE MOVEMENT SWITCH SELECTION

- 0 - NO CHANGE
- 1 - DOWN
- 2 - LEFT
- 3 - RIGHT
- 4 - UP
- 5 - RIGHT CLICK
- 6 - LEFT CLICK

SETUP 1: 3 SWITCH HEAD ARRAY

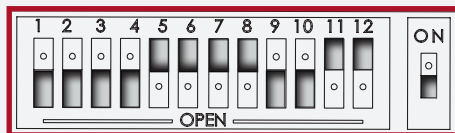


SWITCH SELECTION

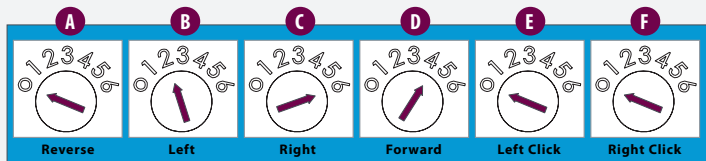


Forward = Up and Down Direction
 Left = Left and Right Direction
 Right = Left Click

SETUP 2: 3 SWITCH JOYSTICK

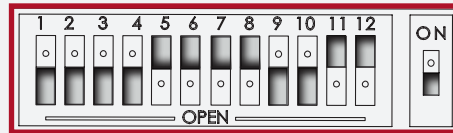


SWITCH SELECTION

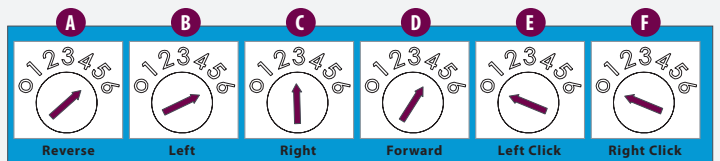


Forward = Up and Down Direction
 Reverse = Right Click
 Right = Right and Left Direction
 Left = Left Click

SETUP 3: 4 SWITCH

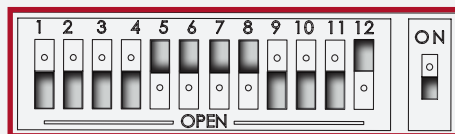


SWITCH SELECTION

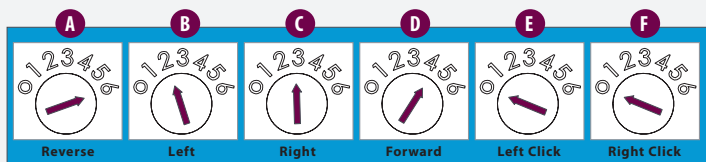


Forward = Up and Down Direction
 Reverse = Left Click
 Right = Right Direction
 Left = Left Direction

SETUP 4: 5 SWITCH

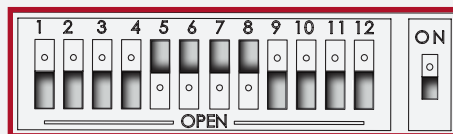

SWITCH SELECTION

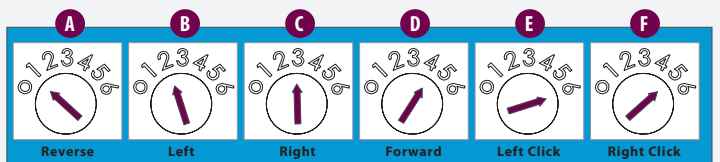

Forward = Forward Direction
 Reverse = Reverse Direction
 Right = Right Direction
 Left = Left Direction
 Right Click and Left Click = Through Switches

